



Vapo Steam[®] Inhaler

USE AND CARE MANUAL



IMPORTANT! READ AND SAVE THESE INSTRUCTIONS

Model V1300-AUS

IMPORTANT! READ AND SAVE THESE INSTRUCTIONS

- ▶ Soothing comfort with gentle vapour therapy
- ▶ Portable – Use anywhere
- ▶ Simply fill with hot water, add 1 capful of Vicks® VapoSteam® or Vicks® VapoPads® and breathe
- ▶ Dishwasher safe - Reusable
- ▶ 5 Vicks® VapoPads® included
Can also be used with Vicks® VapoSteam® and Vicks® VapoSteam® Double Strength Inhalant

Congratulations on your purchase of the Vicks® VapoSteam® Inhaler. Your new Inhaler provides all the benefits of vapour therapy at home, at the office or when travelling.

Directions for Use

Rinse unit out thoroughly before first use.

1. Remove top of unit from base by twisting in opposite directions and lifting off top.
2. Fill base to water line with hot (not boiling) tap water.
NOTE: If you overfill, the unit will leak.
3. Open Vicks® VapoPads® (included) and add to the water or add 1 capful of Vicks® VapoSteam® to the water. Replace the top and inhale steam. Can be used as often as required.
4. Breathe in the soothing Vicks® vapours.



CAUTION: Product contains hot water. To avoid hot water spilling after filling, be sure to properly secure top housing by twisting completely so it snaps back into place.

Care and Cleaning

After each use, thoroughly wash and dry your inhaler. The unit is top shelf dishwasher safe.



Use with Vicks® VapoSteam® Can also be used with Vicks® VapoPads®

Your inhaler was designed to work with Vicks® VapoSteam® or Vicks® VapoPads®. These provide menthol and eucalyptus vapours for soothing comfort.



Vicks® VapoSteam® Vicks® VapoPads®

Recommended Usage for the Vicks Menthol VapoPads (VSP-19):

- Not recommended for infants under 10 lbs (4.5 kgs).
- For children between 10-22 lbs (4.5-10 kgs) do not use more than 2 pads in a 24 hour period.

VapoPad Ingredients: Eucalyptus Oil, Menthol, Glycol, Cedar Leaf Oil & Others.

⚠ WARNING: DO NOT rub face or eyes after touching the VapoPad. Oils in the pad could act as an irritant to eyes.



Distributed by:
Felton Grimwade & Bosisto's Pty. Ltd.
61-81 Clarinda Road (Po Box 74)
Oakleigh South, Victoria 3167, Australia

Free call: 1800 655 841
E-mail: mail@fgb.com.au
Website: www.fgb.com.au

kaz

Kaz USA, Inc., a Helen of Troy Company • Marlborough, MA 01752 USA • www.kaz.com/vicks

Distributed in Australia by Felton Grimwade & Bosisto's Pty. Ltd.

This product is manufactured by Huizhou Huimo Technology Co., Ltd. for Helen of Troy Health and Home, under license from The Procter & Gamble Company. VICKS and other associated trademarks are owned by The Procter & Gamble Company.

© 2018 All Rights Reserved. • Made and printed in China

P/N: 311M1300290R2
Date: 08JAN18